CareMat

mit 869.2125 MHz VarioMobil® Funksender

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Designed in Switzerland / Made in Switzerland

CareMat

with 869.2125 MHz VarioMobil® radio transmitter

(social alarm frequency in accordance with DIN EN 50134)

Pressure-sensitive bedside mat
to detect patients who wander or fall

Translation of the original instructions

1 Safety instructions

- Read these operating instructions thoroughly before putting the device into operation and keep them for future reference.
- Not a safety component in accordance with the EU Machinery Directive must not be used for personal protection or EMERGENCY STOP function.
- Do not use this product other than for its specified application.
- Failure to follow these safety precautions may cause damage to device or objects, serious personal injury, or death.
- Observe all applicable local, national, and international safety standards, codes, and laws.
- After accessing the inside of the device, ensure the cover/protection seal is closed tightly to achieve designated protection rating.
- Warranty claims will be invalid in the event of any damage to the product, to goods or persons that is due to failure to observe these instructions.

Instructions for nursing staff

- The CareMat operates purely as a support for the nursing staff, but is not a substitute for them.
- The functionality of the CareMat is to be checked each time that it is used.

Notes for the installer



- The electrical installation work and commissioning of the product may only be carried out by a qualified technician.
- Avoid touching any electronic components.
- ESD safety rules should be observed.

2 Product description


2.1 Radio system description

The CareMats with 869.2125 MHz VarioMobil® radio transmitters are compatible with the VarioRec® radio receivers from Lehmann Electronic GmbH. The radio receivers (not part of the scope of delivery) can receive the radio signals from the radio transmitters on the social alarm frequency bands approved for this purpose, and relay them to a cable-connected or wireless call system, for example. Detailed information about this device and other compatible social alarm frequency devices can be found on the following web-sites:


www.lehmannweb.de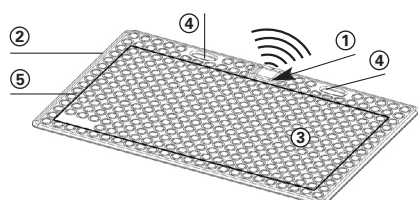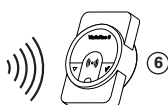

CareMat A01T L869
CareMat A11T L869
1100 x 700 mm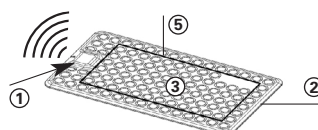

CareMat B01T L869
CareMat B11T L869
700 x 400 mm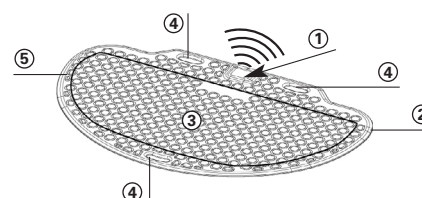


CareMat C01T L869
CareMat C11T L869
1100 x 700 mm,
semi-circular

- ① Housing with radio transmitter
- ② Chamfered, wheelchair-accessible edges
- ③ Non-slip surface with stud texture design


- ④ Carrying handle
- ⑤ Active switching zone
- ⑥ VarioRec® radio receivers (not part of the scope of delivery)

3 Day to day operation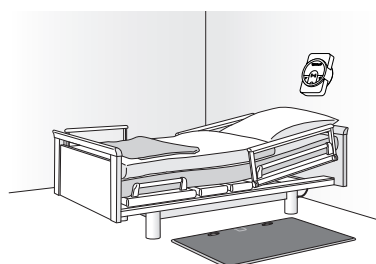

CareMat should be centrally aligned next to the bed so it is impossible to bypass.

Connect the associated radio receiver to the call system.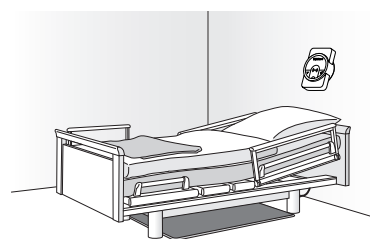

If the CareMat is not needed during the day, it can be stored underneath the bed.

4 Cleaning and disinfecting

- The CareMat can be cleaned using commercially available disinfecting and cleaning agents with max. 70 vol% alcohol. No corrosive agents should be used.
- The CareMat is neither autoclavable nor can it be disinfected using steam.

5 Transport and storage

- Do not bend CareMat during transport, storage or installation.
- CareMat must be stored flat in a clean, dry location with temperature range 0 °C to +55 °C.
- Do not place any heavy loads on the CareMat and do not stack more than 10 packed mats on top of each other.

6 Radio system combined with cable-connected light signalling system

6.1 CareMat configuration


A CareMat with integrated VarioMobil® radio transmitter can be combined with different radio receivers with the same frequency as described in 2.1. Radio receivers do not form part of the scope of delivery. The performance features offered by each radio receiver and the configuration can be found in the technical descriptions provided by the manufacturer, Lehmann Electronic GmbH. These can be found online at www.lehmannweb.de.

A radio receiver must be installed in the call system by a qualified technician.

! Note: Once configured, a CareMat with radio transmitter and a radio receiver form a call system. Particularly when several of these systems are in use in adjacent rooms, the individual systems should be identified clearly to ensure that components cannot be mixed up between rooms.

6.2 Replacing radio transmitter battery

Remove housing cover on rear
(6 countersunk screws M3x5, cross-head size PH1)

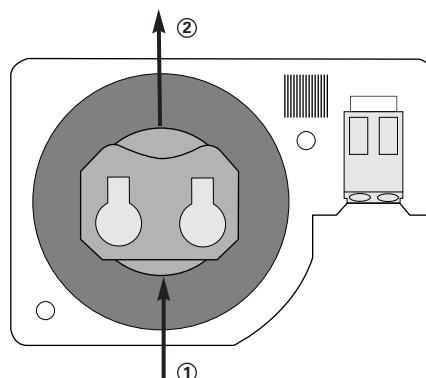- ① With a thin, pointed object (preferably plastic), push the battery out of the housing compartment.
- ② Insert the new battery, ensuring the positive (+) side is facing you.

Reinstall housing cover, carefully tighten the 6 screws by hand, do not overtighten.

7 Maintenance and troubleshooting

The CareMat is maintenance-free until the batteries need to be replaced. Ensure battery compartment screws are not over-tightened.

If batteries have been replaced and the CareMat is still not functioning properly, check the following:

- Is the call system operating and the radio receiver correctly connected?
- Is the CareMat correctly taught in to the radio receiver?
- Is the signal transmitter battery OK?

If the CareMat is still not functioning after the above have been checked, consult a qualified expert.

8 Technical specifications

Article	CareMat Ax1T L869 / Bx1T L869 / Cx1T L869	Radio protocols	Lehmann Electronic GmbH, VarioRec®
Dimensions (LxWxH)	1100x700x9 mm / 700x400x9 mm / 1100x700x9 mm, height with housing 15 mm	Radio frequency (Social alarm frequency)	CareMat xx1T L869: 869.2125 MHz
Weight [kg]	8.2 kg / 3.2 kg / 6.5 kg	Working life	Dependent on application approx. 2 – 3 years
Material	Polyurethane (PU)	Radio range	up to 30 m in buildings, dependent on structural building conditions (up to 100 m in free field)
Colour	Careyellow, Caregrey	Type of battery	1x CR2032, 3 V Lithium
Surface	Stud texture design	Type of protection	IP54
Weight of persons	At least 10 kg	Ambient temperature	0 to +55° C
Chemical resistance	Water (room temp.), body fluids, customary disinfectants, max. 70 vol% alcohol	<i>Subject to technical modifications</i>	

9 EU Declaration of Conformity

CE See attachment

10 WEEE

Devices with this symbol must be treated separately during disposal. This must be done in accordance with the laws of the respective countries for environmentally sound disposal, processing and recycling of electrical and electronic equipment.

11 Contact

BBC Bircher Smart Access, BBC Bircher AG, Wiesengasse 20, CH-8222 Beringen, www.bircher.com

Designed in Switzerland / Made in Switzerland

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